The background of the entire page is a close-up, slightly blurred image of numerous interlocking mounting plates. Most are bright yellow, but there are several dark grey ones interspersed. The plates have various shapes, including circular and triangular, with different patterns of slots and protrusions. The lighting is bright, creating a high-contrast, industrial feel.

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All articulators are optionally available with screw type or MPS magnetic mounting plate system.

The screw type mounting plate system is also used to attach the AXIOSPLIT® system. The user can easily convert using SAM® conversion kits.

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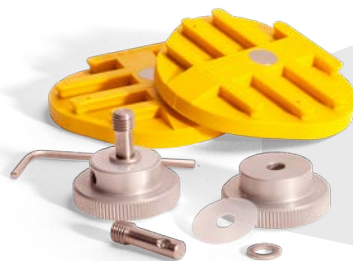
FOR CHANGING OR INSTALLATION YOU WILL

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ART 114

per le basette magnetiche MPS
for MPS Magnetic Mounting Plate System

- 2 porta-magnete (M 5x13mm) con dado (chiuso)
 - 1 chiave, gialla (MPS 101K),
 - 1 aiuto di centratura (MPS 111) e
 - 2 basette gialle con bottone ritentivo
- 2 magnetic holding cups (M 5x13mm) with nuts (closed)
 - yellow wrench (MPS 101K),
 - cup positioner (MPS 111),
 - 2 mounting plates, yellow, with adhesion plates

MPS 101K

ART 120

ASP 420

ASP 5xx

ASP 425

MPS 150

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INSTALLATION OF MPS MOUNTING PLATES

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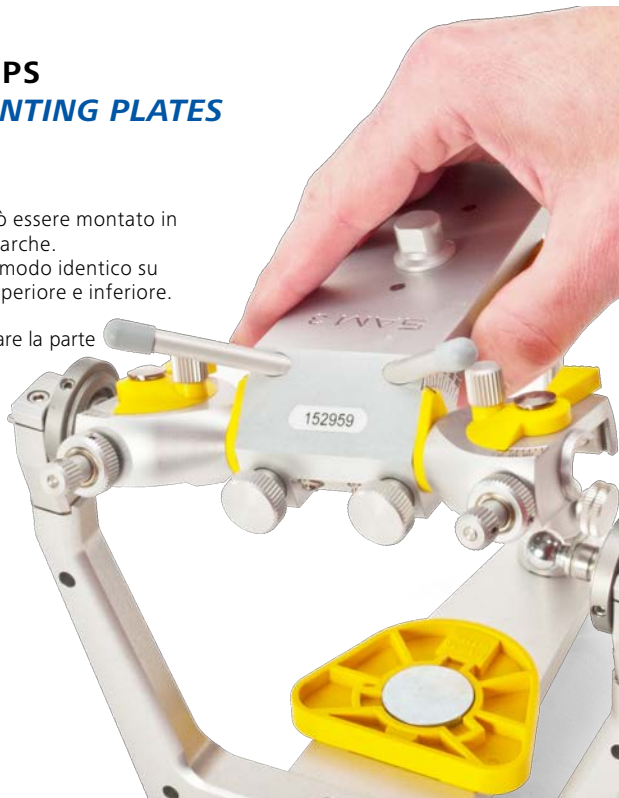
The easy to use MPS magnetic mounting plate system can be installed in all SAM® devices as well as in the devices of other manufacturers. Installation and removal is identical for all SAM® devices, as well as in the upper and lower parts.


To facilitate the assembly, we recommend to separate the upper member of the articulator from the lower.

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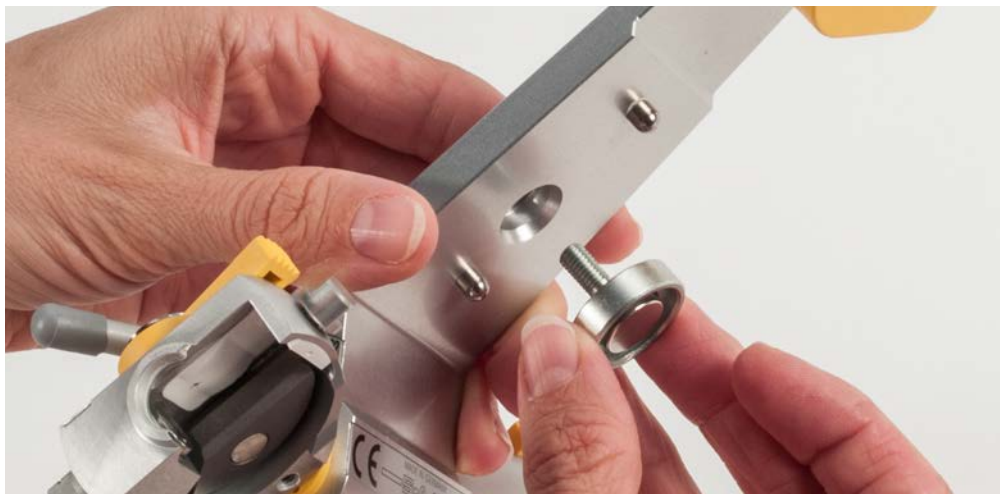Remove the existing mounting plate systems. Instructions for removal are described for the SAM® screw type mounting plates starting on page 32

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Insert magnetic holding cup (MPS 190) into the hole in the upper or lower member of the articulator. The magnetic cup must face inwards.

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Screw the cap nut (MPS 186) loosely onto the thread of the magnetic head (MPS 190).

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Place the MPS mounting plate fully onto the pins and the magnetic cup of the articulator.

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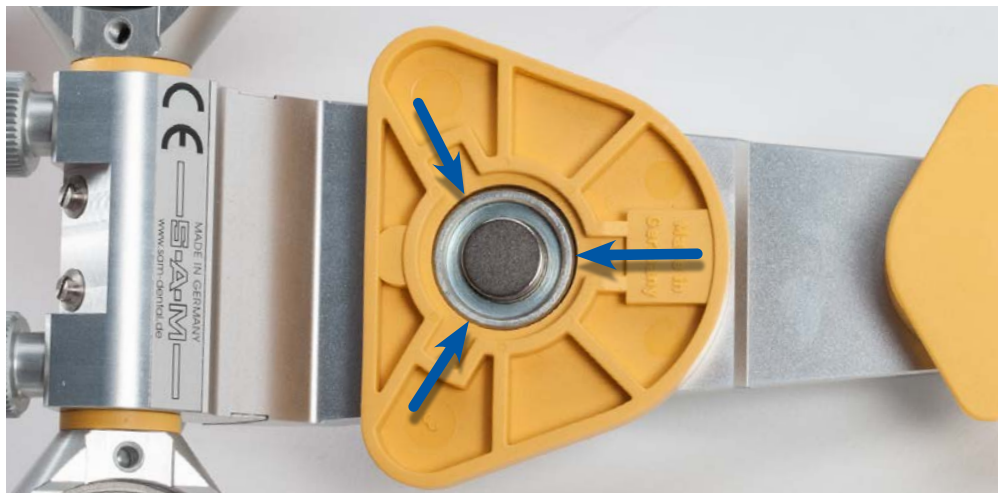
Use cup positioner (MPS 111) to align the magnetic holding cup correctly on to the mounting plate.

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*Tighten the cap nut with the plastic key (MPS 110). **Caution:** Metallic tools can damage the cap nut. Please use the included SAM® wrenches.*

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
*The magnetic mounting plate **must** always be positioned **in center position** related to the cup holder. If not, repeat the installation!*

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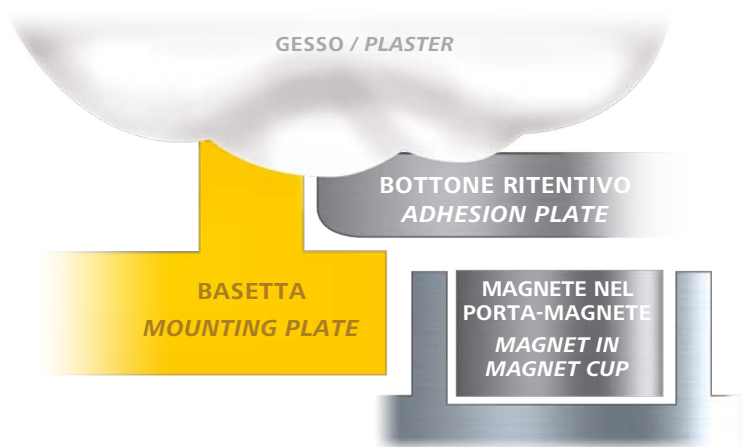Set the adhesion plate with the rounded edge downwards into the provided hole.

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The slightly rounded and shiny side shows downwards when it is inserted into the mounting plate (the rounded edge side shows space for plaster).

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The removal of the MPS system is in principle exactly the opposite of the installation. Also note the use of metal tools as these will damage the cap nut. You can use the safer SAM® wrench.

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Remove the mounting plate from the articulator.

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Loosen the cap nut completely with the plastic wrench (MPS 110) while holding the magnetic cup with the other hand.

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Magnetic plate holding assemblies are reusable.

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INSTALLATION OF MOUNTING PLATES FOR SCREW TYPE SYSTEMS

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The time honored screw mounting plate system can be installed in all SAM® devices. Installation and removal is carried out on all SAM® devices in the same way. Also the installation or removal in the upper and lower parts is identical.

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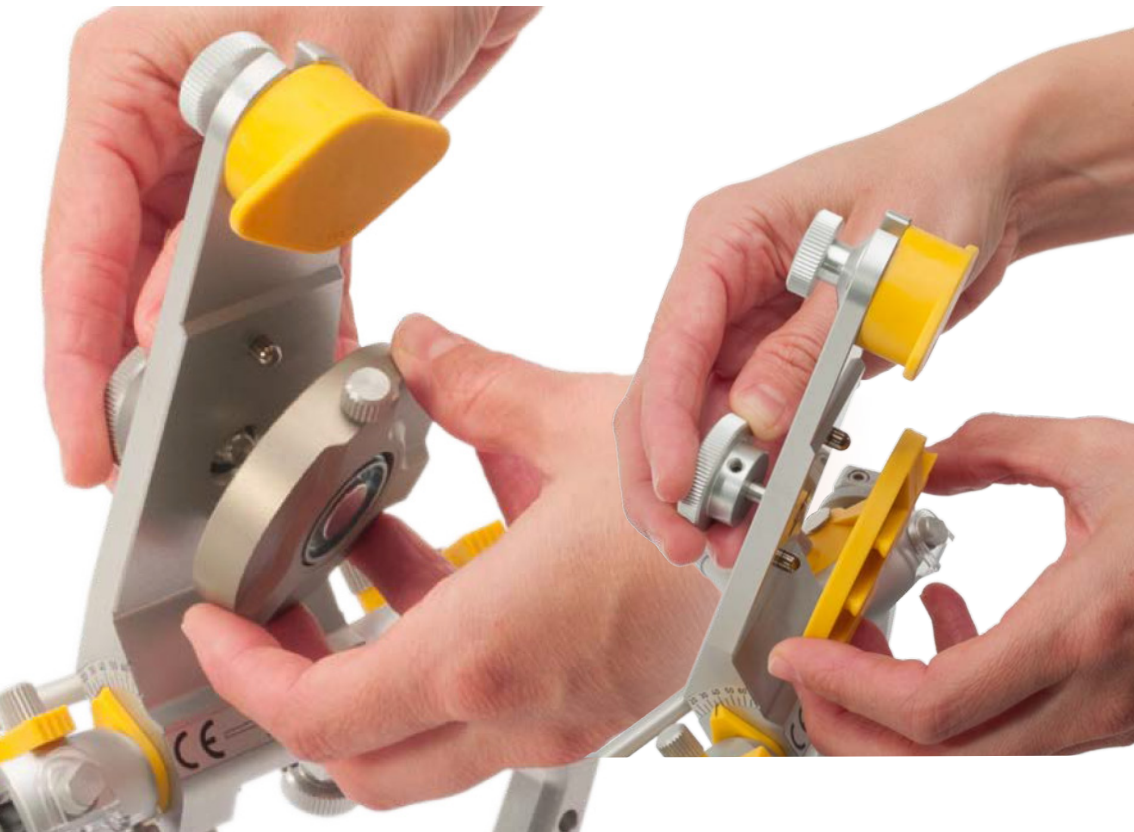
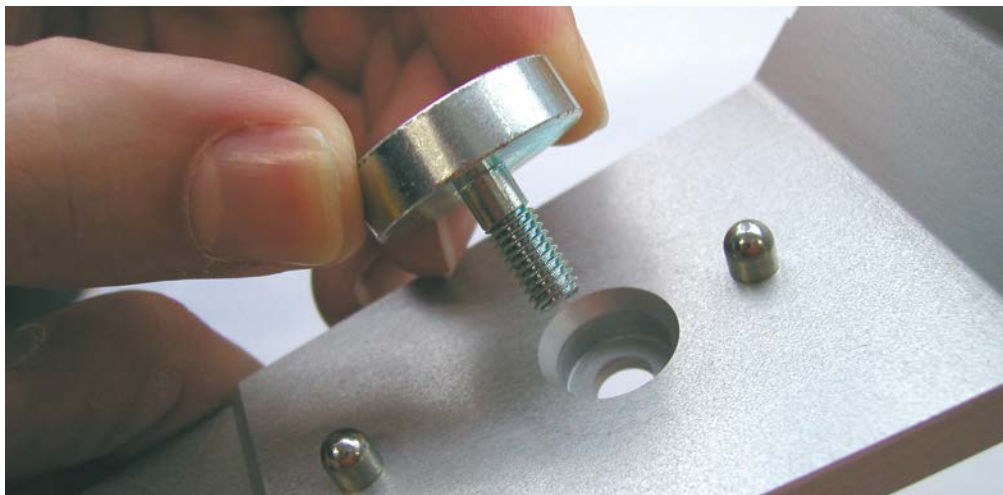
Note:

For the installation of the SAM® AXIOSPLIT® system for Synchronizing several articulators, you also need the screw system.

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
Remove the existing mounting plate systems. Instructions for removal are described for the SAM® MPS magnetic mounting plates starting on page 18.

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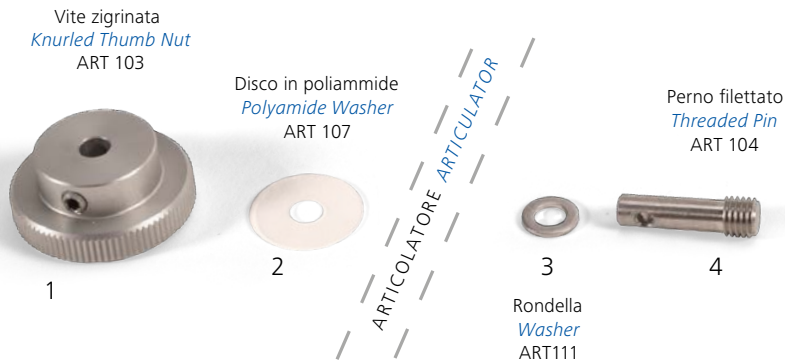The articulator upper member is separated from the lower.

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Take care to the installation sequence of the parts from the outside to the inside: 1 and 2 on the outer side, 3 and 4 on the inner side of the articulator.

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*Slide the threaded bolt (4) with the metal washer (3) through the hole in the articulator.
The thread always shows inwards!*

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Place the polyamide washer (2) over the thread bolt on the top of the articulator. This protects the articulator from scratches.

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Place the knurled Thumb Nut (1) so that the hole in the bolt aligns with the securing screw (arrow).

Stringere la vite a brugola con la chiave esagonale da 2 mm. La vite a brugola collega saldamente la vite zigrinata al perno filettato.

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*Tighten the set screw using a 2mm hex wrench (Allen). The set screw connects the knurled Thumb Nuts firmly on the bolt. **Caution:** The set screw must be completely recessed.*

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The knurled Thumb Nut and bolt are now secured in such a way that they can not come loose from each other. Now either a SAM[®] screw type mounting plate or the SAM[®] AXIOSPLIT[®] system can be screwed onto the inside of the articulator.

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The removal of the screw type system is in principle exactly the opposite of the installation.

After removing, also pay attention to the parts that can be reused elsewhere.

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First unscrew the mounting plate from the articulator.

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Use a 2mm hexagon screwdriver (Allen) to unscrew the set screw completely from the knurled Thumb Nut.

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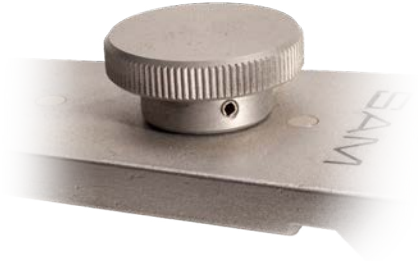
This will release the threaded bolt, which you can now remove from the articulator.

You can reuse the unscrewed parts (Knurled Thumb Nut (1), Polyamide Washer (2), Washer (3) and Threaded Pin (4) in other devices.

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In the case of older SAM® devices (before 1995), the threaded pin was secured by a pressed (and partially glued) conical bolt instead of the screw, which can not be removed so easily.

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Older devices can be retrofitted. In this case, we recommend our repair service, which will make the conversion cost-effective for you. This service of course is also available for all new devices.

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Just ask your local dealer or contact us directly:

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are registered trademarks of the SAM® Präzisionstechnik GmbH
sind eingetragene Warenzeichen der SAM® Präzisionstechnik GmbH
SAM® Präzisionstechnik GmbH